

[back](#)

## MOTomed gracile

### 7 Leg insertion aid

This function allows to position each safety foot shells so that it is most comfortable for you to insert your legs.

This function is activated by pressing the blue button shown in the picture on the right.

Firstly insert your weaker leg. Move the foot shell into position using either the "pedals forward" or "pedals backward" button. After you have inserted the first foot and secured it, press the button again (either "pedals forward" or "pedals backward") to move the other foot shell into a position that is comfortable to insert and secure the second foot.

Note: As soon as you let the "pedals forward" or "pedals backward" button go, the rotation of the foot shells stops.



[recommend this website to a friend](#)

[top of page](#)