

[back](#)

MOTomed letto

1 Operation

Velocity

The operation of the MOTomed letto is safe and easy.


The therapy/nursing staff can set directly all the functions they need for the daily movement therapy: fast or slow, light or heavy, forwards or backwards. Only upon pressing the button "start" the device will start to move the legs slowly and gently. At the same time all functions are activated which are required for the training and ensure the users' safety (MovementProtector, SpasmControl with automatic change of direction etc.)

Adjustabilities for the training:

- Velocity (number of revolutions) during passive training from 0 to 60 revolutions per minute.
- Resistance during active training – from very light up to very heavy (1 to 12 Nm) – can be set in 12 subtle nuances.
- The training duration can be set from 0 to 120 minutes.

Important for the safety of the user:

Velocity and motor power can be adjusted separately.

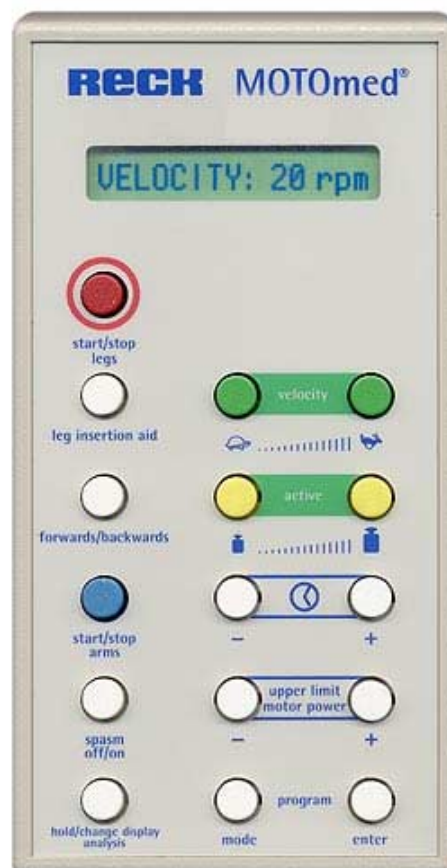

[recommend this website to a friend](#)

[top of page](#)