

[back](#)

MOTomed viva2

7 Leg insertion aid

With this function you can have the foot shells move into position so that you are able to insert the feet as comfortably as possible.

This function is activated by pressing the blue button shown in the picture on the right.

Firstly insert your weaker leg. Move the foot shell into position using either the "pedals forward" or "pedals backward" button. After you have inserted the first foot and secured it, press the button again (either "pedals forward" or "pedals backward") to move the other foot shell into a position that is comfortable to insert and secure the second foot.

Note: As soon as you let go of the "pedals forward" or "pedals backward" button, the rotation of the foot shells stops.



[recommend this website to a friend](#)

[top of page](#)